

DELUXE DIGITAL DENTURES



- ✓ Relineable & Repairable
- ✓ Exceeds ISO High Impact Standards
- ✓ Printed Using Carbon Technology
- ✓ True to Original Design
- ✓ BAM! Technology Strengthens Denture In-Mouth



Prescribe Deluxe Digital Dentures Today!

Visit blackburndentallab.com/send-case/new-doctor to get started!

How to Prescribe a New Denture

1 Step One: Impression

Use current impression technique.

2 Step Two: Model

Blackburn will pour the model and create bite rims based on the provided impression.

3 Step Three: Scan

Blackburn will scan traditional bite rims and the model/impression.

4 Step Four: Digital Design

Creation of digital set-up and design.

5 Step Five: Try-In

Receive the final printed denture base with anterior denture teeth set and bite blocks in the posterior.

Please note: This step can be skipped if you chose to go straight to final denture without try-in.

6 Step Six: Final

Upon completion of try-in or approval from the digital design in step four, Blackburn will fabricate and deliver the final denture.

Prescribe Digital Innovation for Every Case



Replacing a Current Denture

Step One: Impression

Use preferred impression technique, whether that be through traditional materials or an intraoral scan, on denture with wash inside.

What to Send:

Traditional: Denture with wash, opposing model, and bite.

Digital: Scan of existing denture with wash relines, bite scan, opposing scan, and facial photos of high and low lip line.

Step Two: Final

Blackburn will provide the final Deluxe Digital Denture.

Immediate Denture: Traditional

Step One: Impression

Provide a traditional impression.

Step Two: Scan

Blackburn will scan traditional bite rims and the model/impression.

Step Three: Digital Design

Creation of digital set-up and design.

Step Four: Final

Receive final denture from Blackburn



Immediate Denture: IOS File

Step One: Impression

Provide scan of upper and lower arches and bite scan.

For submission, view our DI protocols by visiting www.blackburndentallab.com/send-case/send-digital

Step Two: Digital Design

Creation of digital set-up and design.

Step Three: Final

Receive final denture from Blackburn.

